

# Discover InPouch<sup>®</sup> TF

The Gold Standard Trich test, **InPouch TF**, a nutrient broth media, allows you to collect, culture and detect in one device. With a highly selective proprietary medium for *Trichomonas foetus*, specificity is increased by inhibiting the growth of yeasts, molds and other bacteria.

## PRINCIPALS

- Direct inoculation to media at the POC
- Easy detection and identification

## COST

- Conveniently collect, culture and result in one device
- One-year shelf life at room temperature (18 - 25°C)
  - Nutritive environment and inhibition are viable for one-year from date of manufacture
- Oxygen resistant plastic pouch seals for transport and incubation

## SPECIFICITY

Specially formulated medium is selective for the growth of *T. foetus*, while inhibiting fungi and bacteria

## SPECIMEN

Vaginal swab | Distal penile swab

## RESULTS


- Incubate at 35 - 37°C
- 15 - 30 minutes for presumptive positive
- Evaluate 2 - 4 days for confirmed presence of Trichomonads
  - 98% of positive results will occur within 5 days.

## AVAILABILITY

**10 pack** | 12-011-002  
**100 pack** | 12-011-004

Also available in a PCR compatible transit tube:

**10 pack** | 12-012-001  
**50 pack** | 12-012-002

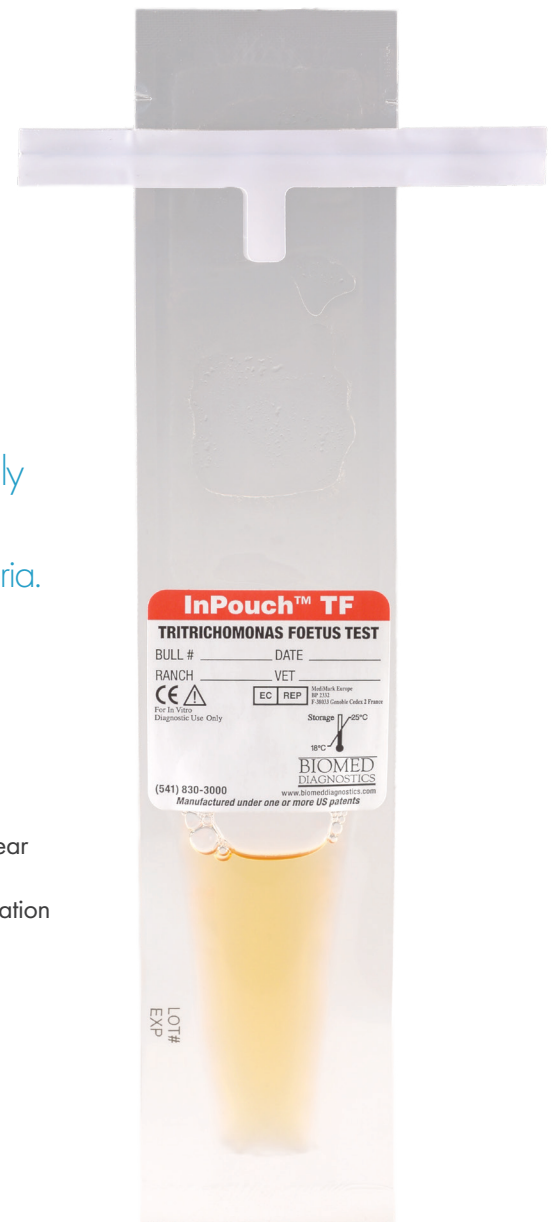

## QUALITY CONTROL:

Test Strain	Result
<i>T. foetus</i>	Growth
<i>P. hominis</i>	Inhibited after 24 hours
<i>Giardia</i>	Inhibited after 24 hours
<i>E. coli</i>	Inhibited
<i>S. aureus</i>	Inhibited
<i>E. faecalis</i>	Inhibited
<i>C. alibicans</i>	Inhibited



Learn more at [biomeddiagnostics.com](http://biomeddiagnostics.com)

**WARNING:** This product can expose you to chemicals including Iron Dextran and Chloramphenicol, which is known to the State of California to cause birth defects or other reproductive harm. For more information go to [P65Warnings.ca.gov](http://P65Warnings.ca.gov).